

# README - IOPI® Report Generator Software

*Software accessory for use with IOPI Pro and Trainer devices - Version 1.0.2.0*

## OPERATING SYSTEM REQUIREMENTS

This application is intended to run on:

- Windows 11 operating system or Windows 10 operating system that is still under support by Microsoft
- Requires .NET Framework 4.8 (if this is not installed by default, go to <https://dotnet.microsoft.com/en-us/download/dotnet-framework/net48> and install the runtime)

## QUICK START

Insert the provided USB drive into your computer and click on: **IOPI® Report Generator Quick Start**.

## VIEW DOCUMENTATION

The full Instructions for Use for the IOPI® Report Generator software are available in electronic form as part of the software. The IOPI® Report Generator README file, User Manual, and License Agreement can be viewed by selecting the **README**, **User Manual**, or **License Agreement** buttons on the Quick Start window.

To obtain a paper version of the Report Generator User Manual, please contact your distributor or IOPI Medical.

## IOPI® REPORT GENERATOR

### Installation

Select the **Install IOPI® Report Generator** button on the Quick Start window and follow the setup instructions.

### Run from USB Drive

This procedure may be necessary if the user is not authorized to install an application directly on a computer. All the program files remain on the provided USB drive.

Select the **Run IOPI® Report Generator From USB** button on the Quick Start window.

## FTDI DRIVER

The IOPI Series 3 devices require a driver made by FTDI. This driver may already exist or may automatically download on a Windows 10 or Windows 11 computer. If you are experiencing connection issues, the computer most likely does not have the correct driver. Follow the instructions below to manually install the FTDI driver.

Select the **Update FTDI Driver** button on the Quick Start window and follow the instructions. Updated drivers may be available from the FTDI website <https://ftdichip.com/drivers/vcp-drivers/>. Note that the “setup executable” version is required.

IOPI® Medical LLC  
18500 156th Ave NE, STE 104  
Woodinville, WA 98072 U.S.A.

Phone: +1 (425) 549-0139  
E-mail: [info@iopimedical.com](mailto:info@iopimedical.com)  
Web: [www.IOPIMedical.com](http://www.IOPIMedical.com)